VisiAid[®]

⊕ SEE BETTER

SWISS MADE

User's manual

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VisiAid[®]

⊕ SEE BETTER

VisiAid[®]

L.E.S.S. SA

Avenue de Longemalle 13

1020 Renens - Switzerland

+41 (0) 21 552 07 10

info@less-sa.com

www.less-sa.com

User's manual

LESS 
⊕ BE BRILLIANT

SWISS MADE

BEFORE USING VisiAid®

By carefully reading this manual, you will be able to fully understand the features of the device and use it to its fullest potential.

By carefully following the instructions provided in this manual, you will be able to maintain the functions of the device over time.

DESCRIPTION OF THE SYMBOLS USED

Do not expose to direct sunlight

Potential risks of injury or health hazards to the user or other people

Do not dispose in general waste

Battery replacement

Types of behavior that may damage the device

Important information

Do not expose to water sources

WARNINGS FOR THE USER

The VisiAid® device is used directly by the user.

The following operations can be performed directly by the user:

- Wear the device.
- Turn the device on and off and change the intensity of the light with the dial.
- Charge the device following the instructions specified in the user manual.
- Clean up the device following the instructions specified in the user manual.

The following operations can only be performed by specialist staff.

The user alone CANNOT perform these operations:

- Dismantle the device.
- Remove screws or replace components.
- Replace the rechargeable battery.

SECURITY WARNINGS

- Do not soak the device.
- Do not expose the device to humidity or steam.
- The power supply provided with the device is similar to normal domestic power supplies; therefore, it does not have any lifesaving function and cannot guarantee electric isolation. The following instructions are recommended to avoid any accidents:
 - * Avoid charging the device in environments that are overly humid, hot, or exposed to the elements.

Danger of strangulation:

- * Do not tighten the charging cable around the neck.
- * Keep the charging cable away from children.

Choking hazard:

- * The charging cable and power supply should be stored out of the reach of children and animals.

INTENDED USE

VisiAid, powered by LuxiBright®, an innovative solution offering optimal short distance shadow-free vision. Enjoy uniform brightness for unmatched clarity. With VisiAid®, rediscover life without constraints, gaining independence in all your activities.

- Ultra-bright: Over 3000 lux at 20 cm.
- Shadow-free: Ultra-uniform illumination.
- User-friendly: Fit the VisiAid® over your glasses for brilliantly illuminated vision.
- Portable: Attach the optional Portable Power Supply and take them with you wherever you go.
- Dimmable: Smoothly select the light intensity desired.
- Comfortable: Lightweight, zero-heat frame designed for extended use.


CONTENT

Content of the package.....	page 6
Items.....	page 6
Introduction.....	page 7
Quick Guide.....	pages 8-11
Table Top Power Supply.....	page 8
Portable Power Supply - Recharge.....	page 9
Portable Power Supply - Set Up.....	page 10
Using VisiAid®	pages 10-11
Cleaning and maintenance.....	page 12
General safety precautions.....	pages 12-13
Proper care and use of the device.....	pages 13-16
Compliance information.....	page 17
Disposal of the product.....	page 17
Warranty.....	page 18
Technical specifications.....	page 19

CONTENT OF THE PACKAGE

- 1) VisiAid®
- 2) Table Top Power Supply (optional)
- 3) Portable Power Supply (optional) with Female adapter
- 4) Charger with plug adapter
- 5) User Manual

ITEMS

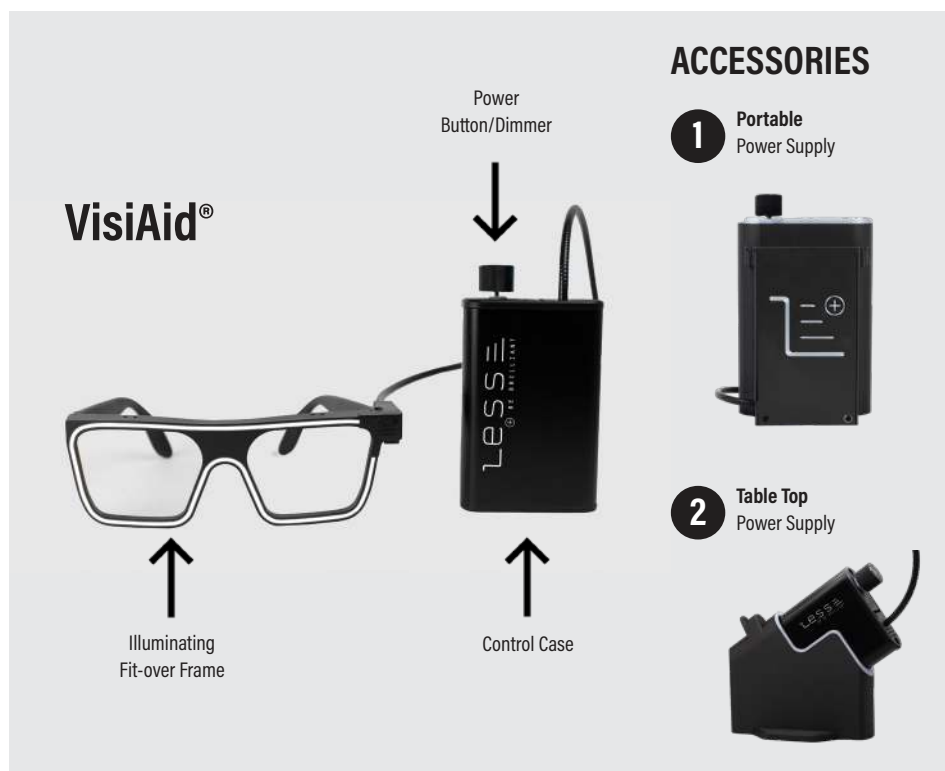INTRODUCTION

VisiAid® is designed for everyday use, providing enhanced close-up visibility and bright lighting. It may improve visual performance in activities such as reading, handling objects and everyday close-up tasks.

There are no contraindications for using the device.

Avoid looking at the light source directly, and do not point it at people, animals, or approaching vehicles.

VisiAid® is equipped with LuxiBright®, a nano-active fiber technology, designed to offer ultra-bright, uniform, and homogeneous light without shadows, providing excellent contrast. The single-button control case allows the use to turn on/off and adjust the intensity of the light.


VisiAid® Lighting Frame

Control Case

QUICK GUIDE

Table Top Power Supply

- 1) Select a flat surface to fix the Table Top Power Supply with the double tape provided. Make sure to have an accessible power outlet at close range for plugging in the charger to the wall. The Table Top Power Supply is intended to be used fixed to a hard and flat surface like an office desk, reading stand or bedside table to prevent involuntary drop downs of the Control Case.
- 2) Insert your country plug adapter into the provided charger (Picture A).
- 3) Connect the charger to the Table Top Power Supply (Picture B).
- 4) Connect the charge to the wall.
- 5) Insert the Control Case into the Table Top Power Supply leaving the "LESS, Be Brilliant" visible (Picture C).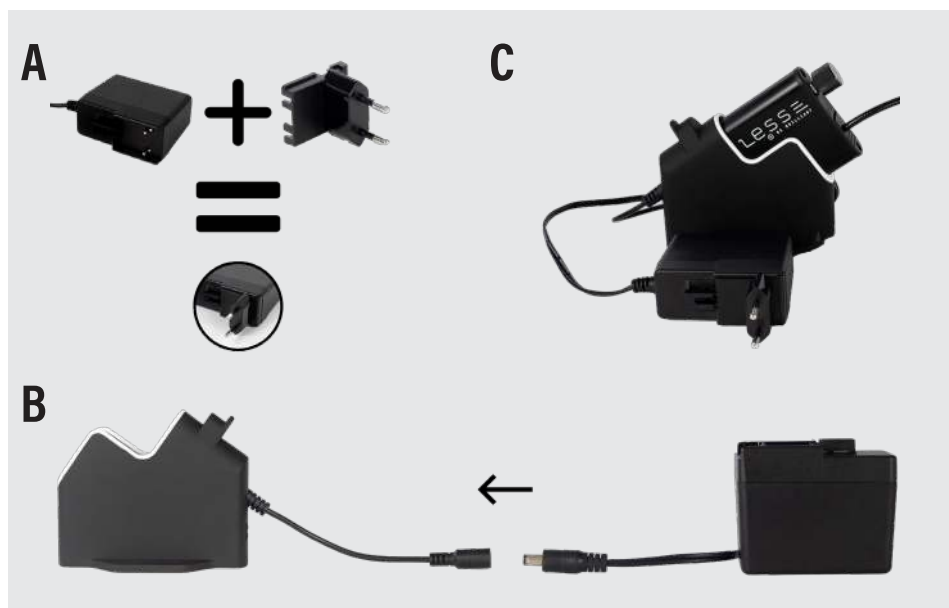

Portable Power Supply - Recharge

Insert your country plug adapter into the provided charger (Picture D).


1. Connect the Portable Power Supply to the charger with the included adaptor (Picture E).
2. Charge the Portable Power Supply by plugging it into the wall. If the battery were fully depleted, it takes about 3 hours to fully charge it.
3. Unplug the Portable Power Supply from the wall saving the adaptor with the charger for re using when you run out of battery.

When not in use, the battery discharges by approximately 1% of its capacity per day. Therefore, after an extended period of non-use, it is recommended to fully recharge the battery.


For battery replacement, please contact the retailer where the VisiAid was purchased.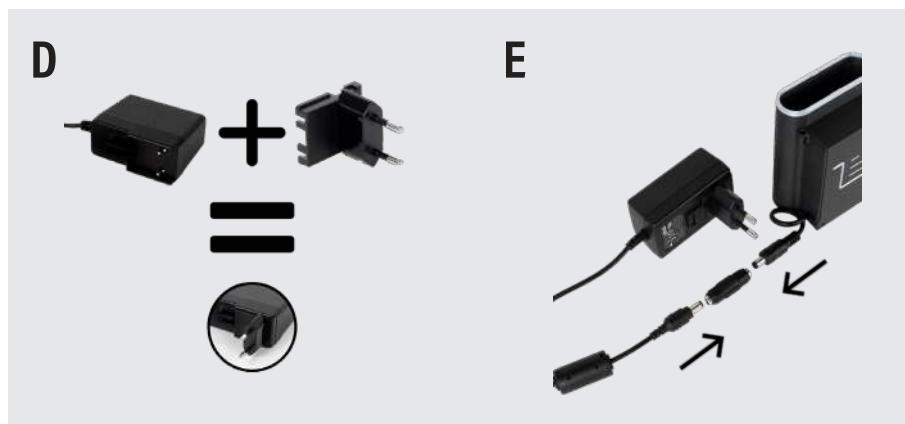

Portable Power Supply - Set Up

1. Insert the Control Case into the Portable Power Supply until you hear a click observing that the power button is on the left side (Picture F).
2. Connect the cable of the Portable Power Supply to the Control Case through the gray hexagonal hole (Picture G).
3. Clip-on the Portable Power Supply onto your belt or trousers.

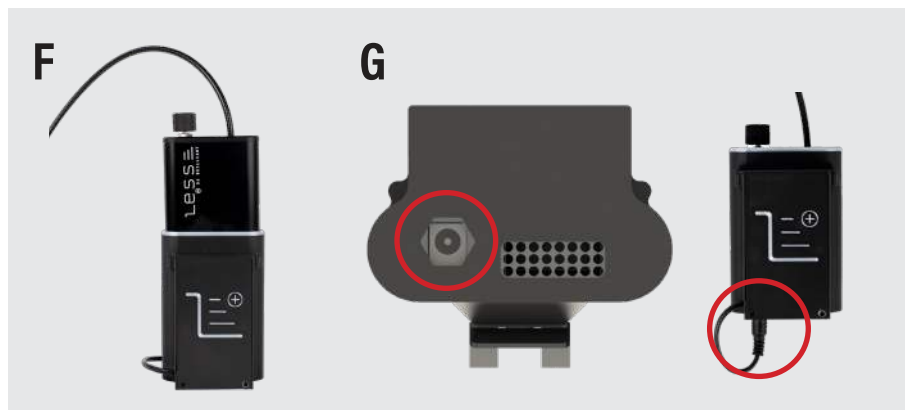If the light blinks, it means the battery is nearly depleted. Recharge the Portable Power Supply (page 9).

You can use the Portable Power Supply for a duration ranging from 1 hour 30 minutes to 6 hours depending on the selected intensity.

Using VisiAid

1. Set up: Fit the VisiAid® ultra bright illuminating frame over your prescription glasses (Can also be wear without glasses).
2. Switch On/Off: Click the button once to turn the VisiAid on and again to turn it off. You will hear a click when correctly pushed (Picture H).
3. Intensity selection: Turn the button anti-clockwise to increase or clockwise to decrease the intensity of the illumination (Picture H)

H

1. Press to Switch on/off

2. Turn to adjust light intensity

Use only the power supply provided with your VisiAid® or sold separately as a VisiAid® accessory. The use of any other power supply will void the manufacturer's warranty.

When you wear the VisiAid®, the optimal distance of the face from the object to be viewed varies from person to person.

Never leave the VisiAid® unattended.

Once you have finished using the VisiAid®, turn off the system by pressing the button on the control case and store it in a dry area, away from direct sunlight and heat sources.

The VisiAid® can be used while standing, sitting, or lying down. The system allows the user to assume natural and body-friendly positions. However, ensure that the various openings of the control case are not obstructed to guarantee proper ventilation of the unit.

CLEANING

For cleaning the components, please use a soft microfiber. The use of cleaning products, alcohol, or other chemicals is not recommended.

Always clean the VisiAid® when it is turned off and disconnected from the power supply.

Failure to follow the use and maintenance instructions stated in this manual may invalidate the product warranty. Please follow the instructions carefully to take full advantage of the warranty conditions.

Do not use any liquid products.

MAINTENANCE

Do not open the VisiAid® components for any reason. Failure to follow the use and maintenance instructions stated in this manual may invalidate the product warranty. Please follow the instructions carefully to take full advantage of the warranty conditions.

For technical assistance, please contact the retailer where the VisiAid® was purchased.

GENERAL SAFETY PRECAUTIONS

Before using the device, carefully read all the information to avoid injuring yourself and others or damaging the device itself.

Do not tamper with, disassemble, modify, or repair the device.

- Any change or modification to the device may void the manufacturer's warranty.

For assistance, please consult your retailer.

- Do not unscrew any screws or open any device cover.
- Do not remove or perforate the battery.
- L.E.S.S. SA is not responsible for the safety of the user in the event of tampering with the product and does not guarantee the operation of the product in this case.

GENERAL SAFETY PRECAUTIONS

Preventing electrical shock, fires, and explosions.

- Do not use damaged power cables or electrical plugs, and do not pull the sockets.
- Do not touch the power cable with wet hands, and do not disconnect the Table Top Power Supply or Portable Power Supply by pulling the cable.
- Do not bend or damage the power cable.
- Do not short-circuit the Portable Power Supply.
- Do not drop or subject the Table Top Power Supply, Portable Power Supply and/or VisiAid® to shock.
- Use only the power supply provided with your VisiAid®, or sold separately as a VisiAid® accessory. The use of any other power supply will void the manufacturer's warranty.
- Do not wet the device, and do not touch it with wet hands under any circumstances.
- Do not connect the Table Top Power Supply or Portable Power Supply to the electrical grid during storms.
- Never position the device on or inside heating devices such as microwave ovens, ovens, or radiators.
- Always turn the device off in potentially explosive environments.
- Always comply with standards, instructions, and indications provided for potentially explosive environments.
- Never use the device near fuels or flammable chemicals.
- Do not leave or carry flammable liquids, gases, or explosives in the same compartment in which the device or its related parts and accessories are located.

Never expose the device to rain or, in general, to liquid substances.

Water and liquids, in general can generate electrical short-circuits, which may potentially cause damage to property or people.

Exposing the VisiAid® to liquids may cause irreparable damage to the device's electrical circuits.

PROPER CARE AND USE OF THE DEVICE

Follow these simple instructions to make the best use of your VisiAid® and protect it from accidental damage.

Protecting the VisiAid®, the Portable Power Supply and Table Top Power Supply.

- Do not expose any of them to extremely hot or cold temperatures.
- Extreme temperatures can cause deformation of the device components and reduce the charging capacity and life of the VisiAid®, Portable Power Supply and Table Top Power Supply.
- Never use damaged battery power supplies.

The fiber must never be disconnected from the control case or cut. In the case of fiber breakage, the laser source will automatically shut down. The power supply must then be switched off and the device returned to the reseller.

Keep the device dry

- Moisture and all types of liquids can damage parts of the device or the electronic circuits.
- Do not turn the device on if it is wet.

Do not use or store the device in dusty or dirty areas.

- Dirt may cause the device to malfunction.

Do not store the device on sloping surfaces.

- The device may fall and be damaged.

Do not store the device in areas that are too hot or cold.

- The device can explode if left inside a closed vehicle or in other closed environments where high temperatures are reached.
- Do not leave the device in direct sunlight for long periods of time (for example, on the dashboard of your car).

Do not drop or hit the device.

- If it is bent or deformed, the device may be damaged and the internal components may not work properly.

Do not point the light towards the eyes of people or animals.

- Pointing the light close to the eyes may cause a temporary loss of vision and/or injury to the eyes.
- Never look directly at the light source.

Preserve maximum battery and power supply life of the Portable Power Supply.

- Avoid keeping the battery under charge for more than one week. Overcharging them can shorten their lifetime.
- Over time, unused battery will discharge and must be recharged before use.
- When the Portable Power Supply is charged and not in use, disconnect it from the power supply.
- Only use batteries, power supplies and accessories approved by L.E.S.S. SA.

Using a generic battery or power supply voids the manufacture warranty and may reduce the life of the device or cause it to malfunction.

- L.E.S.S. SA is not responsible for the user's safety when using accessories or products not approved by L.E.S.S. SA.

Do not bite or chew on the device.

- This may cause damage to the device or even lead to explosions and injuries. Always supervise children using the device.
- Do not put the device or supplied accessories into your eyes, ears, or mouth. This may cause suffocation or serious injury.

Do not use this device while driving a vehicle.

- This aid is designed for close-up activities like reading, cooking, or manual work.
- Always pay attention to your surroundings to avoid injuring yourself or others.

Do not disassemble, modify, or repair the device.

- Any change to the device may void the manufacturer's warranty. For assistance, please consult your retailer.
- Do not remove or perforate the battery.

Do not paint the device or put stickers on it.

- Paint and stickers can block the moving parts and prevent proper operation. The materials the product is made from are biocompatible. However, in the case of allergies to paints or metal components of the product, itching, eczema or swollen skin may occur. If this happens, stop using the product and consult a doctor.
- When cleaning, use a soft microfiber cloth to clean the device or battery power supply. Do not use chemical agents or detergents.

This device is not a toy.

- Do not allow children to play with the device, as they could hurt themselves or others. They may also damage the device.

Do not point the light towards the eyes of people or animals.

Only qualified personnel should carry out maintenance operations on the device. Please refer to your retailer for more information.

- If maintenance on the device is carried out by unqualified personnel, they may damage the device, and subsequently void the manufacturer's warranty.

The following table shows the various troubleshooting measures for L.E.S.S. SA systems:

Type of Failure	Probable Cause	Solution
Light off	The unit is not running	Turn on the device (page 11)
	Battery is dead	Charge the portable power supply (page 9)
	No main power	Insert correctly the male plug into the control case through the gray hexagonal hole (Page 10)
	No main power	Change the main plug
	The control case is broken	Contact your retailer for instructions
Flash at startup	The light fiber is broken	Contact your retailer for instructions

If you are unable to fix the issue by following the measures mentioned above, please contact your retailer to get further assistance.

COMPLIANCE INFORMATION

The VisiAid product is designed and manufactured according to the highest quality standards. The appositeof the CE Class 1 MD, FCC marking is an evidence that the product has been tested and certified according to European and North America regulations. Additionally, being ISO 9001 certified, demonstrate our commitment to excellence and regulatory compliance in all aspects of our product's design and manufacturing.

DISPOSAL OF THE PRODUCT

Do not dispose of the product with ordinary waste.

The VisiAid®, Portable Power Supply and Table Top Power Supply contain electronic components. It therefore cannot be disposed of with normal domestic waste. Contact an authorised service centre for safe disposal.

DISPOSAL OF THE PORTABLE POWER SUPPLY

Take off the control case from the Portable Power Supply and dispose as special waste, at the collection points indicated for recycling lithium-ion battery.

For further information on how to dispose of used battery, please contact your retailer or your local waste disposal service.

Never dispose the device by throwing them in the fire. Follow current laws for their disposal.

WARRANTY

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual regarding faults arising in connection with manufacturing defects or material defects, under normal operating and maintenance conditions. In the event of damage due to improper handling, including damage from a fall or impact, abuse and poor use, tampering or modification, no warranty claims can be made. Consumables, including the battery, are excluded from the warranty.

Warranty claims are only accepted upon presentation of the sales receipt, with mention of the purchase date, retailer name, model and serial number, and are to be presented to the retailer where it was initially purchased.

This guarantee does not modify or extend the rights of professional users established by law, nor those that the professional users may have in relation to retailers as a result of sales contracts stipulated with the retailers themselves. Any responsibility resulting from indirect damage or lost profits is to be excluded.

TECHNICAL SPECIFICATIONS

Name	VisiAid®
Reference	VA-HF
Color Temperature	5400°K
Illumination	>3'000 lx at 20 cm
Light Intensity Control	Dimmable from 20 to 100%
Frame Opening Size	6 cm x 4,5 cm
Frame Weight	50 g
Accessoires	Table OR Portable Top Power Supply
Cable Length	100 cm
Operating Temperature	0-40°C**
Table Top Power Supply Size	11,12 cm x 5,6 cm x 9,1 cm
Table Top Power Supply Weight	153 g
Portable Power Supply Size	7,6 cm x 5,6 cm x 11,05 cm
Portable Power Supply Weight	276 g
Battery	Included in the Portable Power Supply
Battery Autonomy	From 1h30 to 6h00, depending on the intensity*

*All statements regarding battery life depend on settings and environmental factors; actual results may vary.

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